

## Self-Collection Paternity Test Instructions

\*\*\* *The results from a self-collected case are not admissible in court* \*\*\*

**ALLTESTS NORTH  
AMERICA  
800.833.0680**

**PLEASE READ ALL INSTRUCTIONS THOROUGHLY BEFORE CONTACTING OUR OFFICE. ALL ANSWERS TO YOUR QUESTIONS SHOULD BE ANSWERED WITHIN THIS DOCUMENT. FAILURE TO FOLLOW INSTRUCTION MAY RESULT IN SAMPLE FAILURE. ALLTESTS IS NOT RESPONSIBLE FOR SAMPLE FAILURE DUE TO IMPROPER COLLECTION.**

### Authorization Instructions

1. *Complete the enclosed Authorization Form. We must have all information to process your test, HOWEVER WE **DO NOT** NEED SOCIAL SECURITY OR ID NUMBERS FOR THESE TYPE OF TESTS.*
2. *If you want your results as soon as they are ready, be sure to complete the report faxing information!*

### Swab Collection Instructions

***Only collect one person at a time, to prevent accidental sample switching  
Never allow contact between swabs from different individuals***

1. *Fill out all forms to the best of your ability. If there is something you are not sure about call our customer service hotline*
2. *Collect DNA samples using only sterile cotton swabs. Regular q-tips are acceptable.*
3. *If swab has two cotton heads, then cut off the end of the swab that will not be used for collection.*
4. *In a brisk circular motion, rub the swab along the inner cheek while twirling the swab generously throughout the mouth. Use 1 swab for each inner cheek.*
5. *Place the swabs in a PAPER envelope only. Do not touch cotton tip once sample has been collected.*
6. *Label each envelope clearly with donor name. Only one set of swabs per envelope*
7. *Mail all completed forms and swab envelopes in a return envelope to:*

**ALLTESTS DNA TESTING  
681 MAIN STREET  
ROCKWOOD. PA 15557**

**ALLTESTS HIGHLY RECOMMENDS USING SOME FORM OF TRACKABLE SHIPPING IN THE EVENT YOUR SAMPLE GETS LOST IN TRANSIT.**