

SELF-COLLECTION PATERNITY TEST INSTRUCTIONS

*** *The results from a self-collected case are not admissible in court* ***

**ALLTESTS
INTERNATIONAL**
800.833.0680

Authorization Instructions

1. Complete the enclosed Authorization Form. We must have **all** information to process your test, *including the signature of a witness to the collection process.*
2. If you want your results as soon as they are ready, be sure to complete the report faxing information!
3. The "Witness" must be an objective 3rd party with no interest whatsoever in the outcome of the test.

Payment Instructions

Any amount you have pre-paid will be credited to your account.

Swab Collection Instructions

***Only collect one person at a time, to prevent accidental sample switching
Never allow contact between swabs from different individuals***

1. Fill out all forms to the best of your ability. If there is something you are not sure about call our customer service hotline
2. Collect DNA samples using only sterile cotton swabs. Regular q-tips are acceptable.
3. In a brisk circular motion, rub the swab along the inner cheek while twirling the swab generously throughout the mouth. Use 1 swab for each inner cheek
4. Place the swabs in a PAPER envelope only. Do not touch cotton tip once sample has been collected.
5. Label each envelope clearly with donor name. Only one set of swabs per envelope
6. Ship all completed forms and swab envelopes to our office for immediate processing. **MAKE SURE TO INCLUDE PAYMENT BY CHECK OR MONEY ORDER.** Credit card payments should be called into our office **at 800.833.0680**
7. Shipping address can be found at the bottom of this page